

Pelican Pool & Spa Hoist Battery Care



Battery LED

LED 1 - LED 2 - LED 3	LED state (Not listed = off)	States in normal use
	LED 1 - 3 constantly on	75 - 100% SOC
	LED 1 + 2 constantly on	50 - 75% SOC
	LED 1 constantly on	< 50% SOC
	LED 1 left side, switches from green to yellow and flashes slowly	Two cycles left, buzzer active
	LED 1 - 3 constantly on	90 - 100%
	LED 1 + 2 constantly on LED 3 flashes slowly	65 - 90%
	LED 1 constantly on LED 2 flashes slowly	40 - 65%
	LED 1 flashes slowly	0 - 40%
	LED 1 + 2 + 3 flash slowly	Charging stopped due to low battery temperature, high battery temperature or other error conditions
	No light in LEDs	Charging stopped due to lost communication to battery



ParaMobility

Making everyday living easier

Phone 1300 444 600

Pelican® Installation & Service
 Delivery, installation + servicing Australia wide
enquiries@paramobility.com.au
www.paramobility.com.au