

# Pelican Pool & Spa Hoist User Guide



## INSTALLATION AT POOL DECK (MOBILE VERSION)

- Ensure the locking pin is engaged before transporting to stop unit from swivelling.
- Wheel the unit to the pool deck and place the base plate over the anchor points. Ensure the wheels of the hoist are placed facing away from the pool.
- Insert the Eye Bolts into the holes and hand tighten only.
- Unlock the locking pin.

## USE OF THE LIFTER

- Always test the lifter by pressing up and down on the hand control before commencing a lift.
- Check safe working load (SWL) of lifter before attempting a lift and make sure the weight of person being lifted complies.
- Using the hand control, raise the lifting arm above the clients head and wheel the chair under the lifting arm, if using a sling refer to sling instructions.
- Gently swing the arm over the water and lower the hoist until the user is safely in the water.

## AFTER USE CARE

- Engage the locking pin to prevent the unit from swivelling.
- Wash the Pelican with fresh water (not chlorine).
- Lower arm to the lowest position and put on the cover.
- Remove the Eye Bolts from the base plate and place them in the travel holes.
- Stand behind the wheels and hold the handle, pull the unit back until travel balance is achieved.
- With care, wheel the unit to the storage area.



Locking Pin

## Emergency STOP



## GETTING STARTED

- **Install the battery** – The Battery should always be removed and replaced into the charger after use. Do not remove the hand control from the lifter as this will disrupt the control box mechanisms. To remove battery use thumb and index/middle finger to push buttons on battery sides and pull out (1 & 2). To mount, grab the battery on sides and steer the battery base over steering pin then push in place (3 & 4).



**Para Mobility**

Making everyday living easier

**Phone 1300 444 600**

Pelican® Installation & Service  
Delivery, installation + servicing Australia wide  
enquiries@paramobility.com.au  
www.paramobility.com.au

**Never push on the actuator to rotate the lifting arm. It will damage the lifting mechanism.**



### BATTERY CARE

- After use, the battery should be placed in the charger to ensure it is fully charged when required.

### EMERGENCY STOP


- Push the red button on top of the battery to activate the emergency stop (1).
- To release the emergency stop take off the battery (2 & 3) and replace it back into the unit again (4 & 5).



**Always ensure Eye Bolts are in place before using (mobile version)**

### EMERGENCY MANUAL SAFETY LOWERING OR LIFTING

- In case of an emergency, use the actuator to activate the manual lowering/lifting function. Rotate the grey or red knob on the top of the actuator clockwise to lower the arm of the hoist.

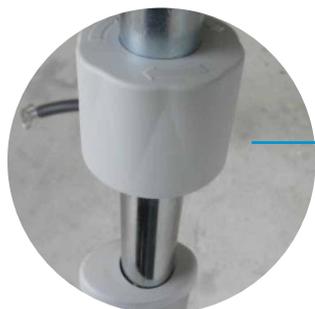

**Manual Emergency Lowering**



**Ensure locking pin is in place before transporting (mobile model)**

